




Digital Accelerator

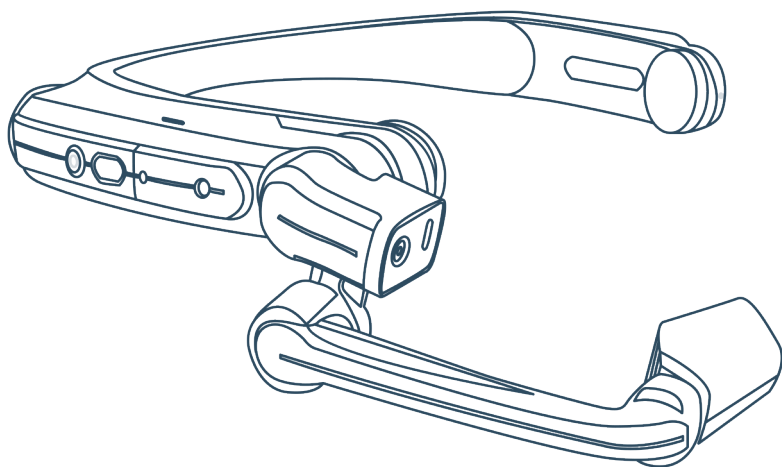

RealWear HMT-1

Getting Started


XpertEye Advanced

amaxperteye.com

A. HMT-1


Before using the XpertEye solution, make sure the device (smartphone or tablet) is charged. For more information refer to section C - "Exit and charge the XpertEye solution".

1

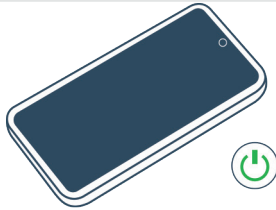


Turn on the device.

2

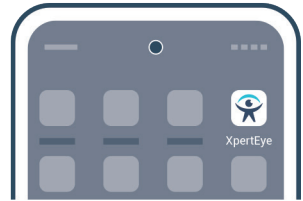


Launch the XpertEye Advanced application.

3

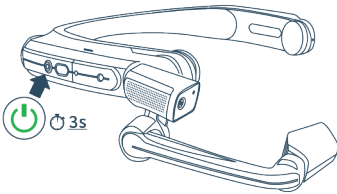


Turn on the HMT-1 by pressing the power button for 3 seconds.

4

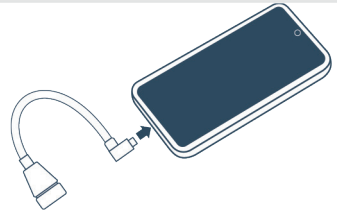


Connect the USB/USB-C (female) cable to the device.

5

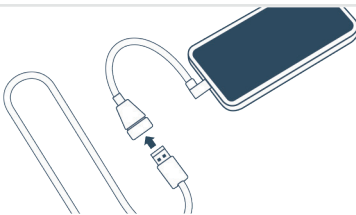


Connect the USB/USB-C (male) cable and connect it to the female connector.

6

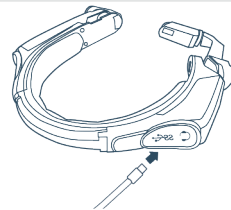


Connect the USB-C connector to the HMT-1.

7

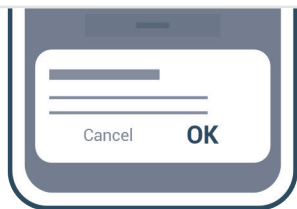

Once the HMT-1 is connected, go to the XpertEye Advanced application, and accept the Android authorization requests to use the HMT-1 camera.

8

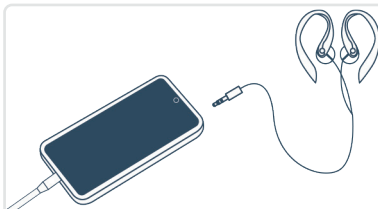

Connect the earphones to the device.

B. HMT-1, Holster and Hub Battery*


Before using the XpertEye solution, make sure the device (smartphone or tablet) and the hub battery are charged. For more information refer to section C – “Exit and charge the XpertEye solution”.

1




Turn on the device.

2

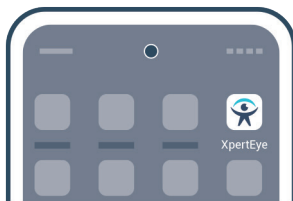

Launch the XpertEye Advanced application.

3

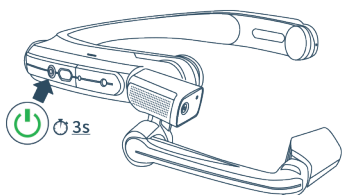

Turn on the HMT-1 by pressing the power button for 3 seconds.


4



Turn on the battery's hub mode by double pressing the "on" button.


Note: A blue light indicates that the hub mode is on.

5

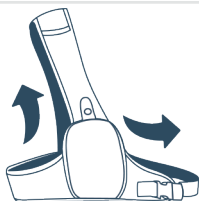


Unfold the holster.

6

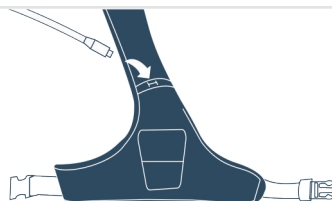


Pass the USB-C cable through the holster, from back to the front.

7

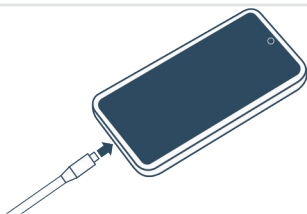


Connect the USB-C cable to the device.

8

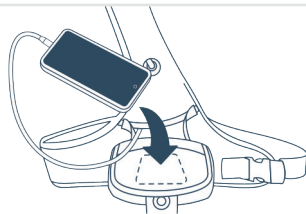


Attach the device to the holster pocket.

9

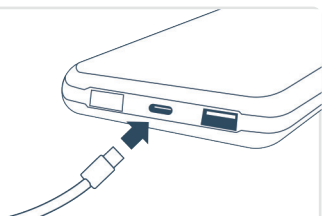


Connect the USB-C cable to the hub battery.

10

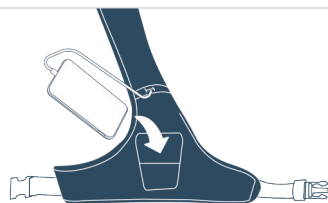


Put the hub battery in the back pocket.

11

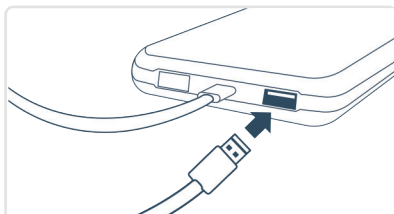


Take the USB/USB-C cable and connect the USB connector to the hub battery.

12

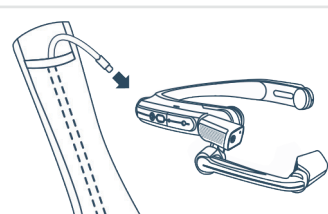


Pass the USB-C cable through the holster from the bottom up and connect to the HMT-1.

13

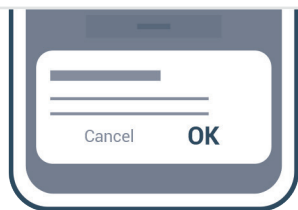


Once the HMT-1 is connected, go to the XpertEye Advanced application, and accept the Android authorization requests to use the HMT-1 camera.

14

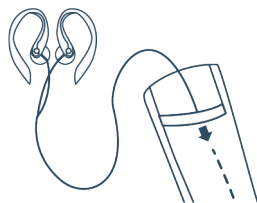

Pass the earphones through the holster, from top to the bottom.

15

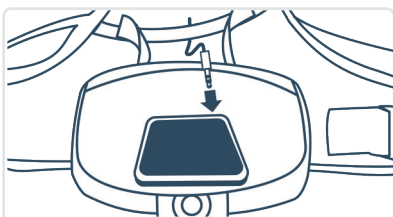


Connect the earphones to the device and zip the holster.

*** The holster and the hub battery are optional and may not be included in your XpertEye Advanced kit.**


C. Exit and charge the XpertEye solution

1

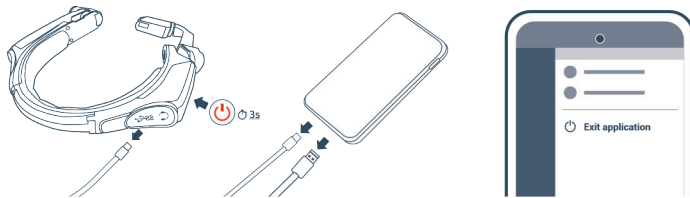

- Unplug the HMT-1 and press 3 seconds the power button.
- Unplug the hub battery.
- From the XpertEye menu, press **Exit application**.

2

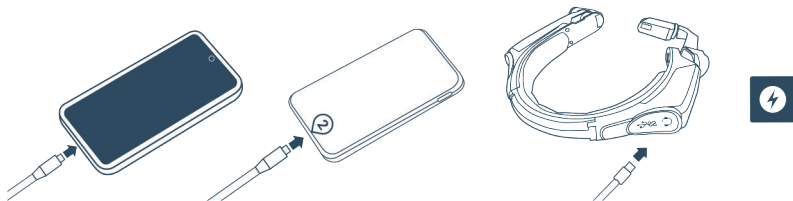

- Charge the device with the dedicated charger.
- Connect the USB-C cable to the battery hub (port n°2) and charge it using a wall charger.
- Repeat this operation with the HMT-1 ($\approx 1h30$).

